


Installation Manual and Operating Instructions

IB-8300, 8500, 8500R

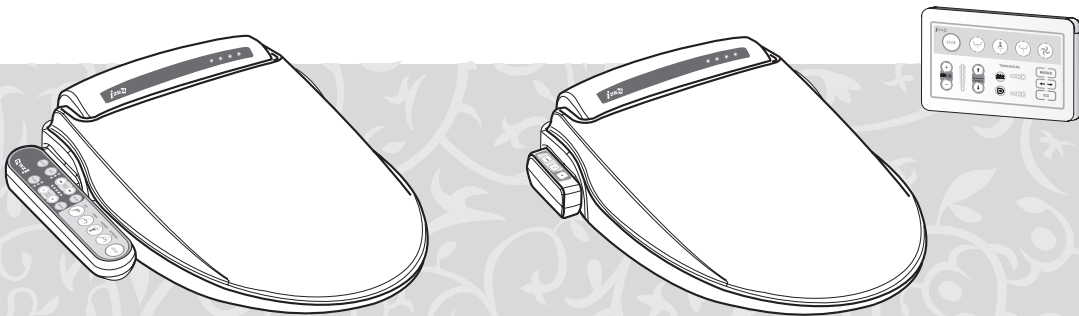

Safety instruction

1. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
2. Children should be supervised to ensure that they do not play with the appliance.
3. If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
4. The new hose-sets supplied with the appliance are to be used and that old hose-sets should not be reused.

-Warning symbols comment:




Do not sit on cover



Do not stand on cover

- This Product should be installed according to this manual.
- Please read this manual carefully and install properly.
- Please keep this manual for future reference.
- Please register Warranty for proper service.

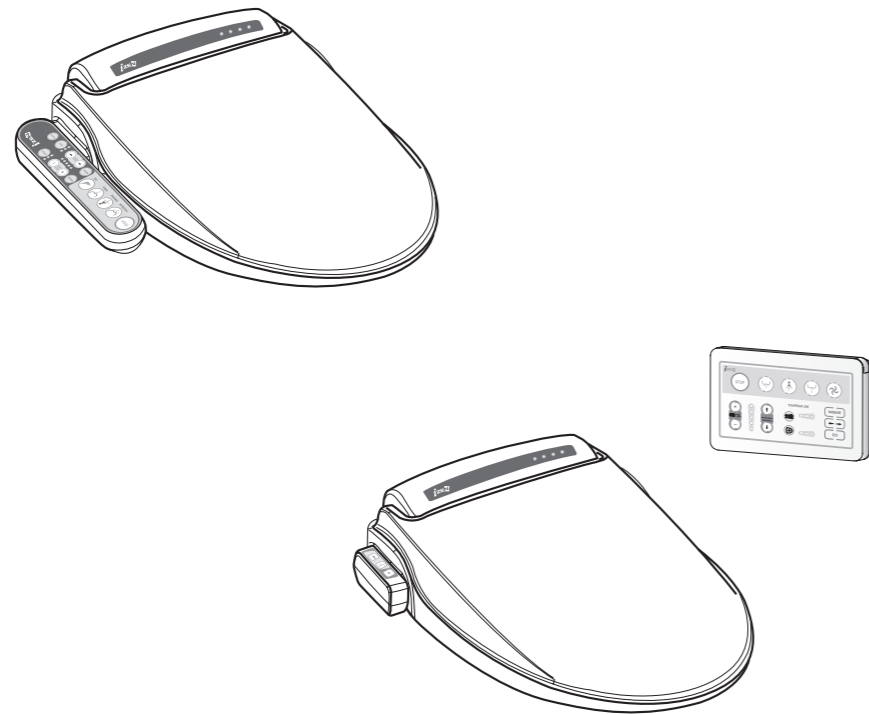

Thank you for purchasing 'I-ZEN' Shower Toilet Seat (Electric Bidet).
This unit was manufactured for household use.

Contents

- IMPORTANT SAFEGUARDS / 4
- GENERAL CAUTIONS
- CAUTIONS BEFORE INSTALLATION 6
- CARTON CONTENTS / IDENTIFICATION 8

- OPERATION (BASIC) 9
- OPERATION (DETAILS) 10

- INSTALLATION 14
- SEPARATION OF THE UNIT FROM THE TOILET BOWL 17

- MAINTENANCE 18
- WATER DRAINING 19
- TROUBLE SHOOTING 20
- SPECIFICATIONS 21
- WARRANTY 22
- INFORMATION TO THE USER 23

PREPARATION

OPERATION

INSTALLATION

MAINTENANCE


I-ZEN Bidet


1 IMPORTANT SAFEGUARDS / GENERAL CAUTIONS


● READ ALL INSTRUCTIONS BEFORE INSTALLATION AND USE


The following terms are used in this manual to indicate hazards of varying degrees of severity


DANGER	Indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Indicates potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Indicates potentially hazardous situation which, if not avoided, may result in minor or moderate injury or in damage to the product or other property.


 **DANGER** - To reduce the risk of electrocution


 **WARNING** - To reduce the risk of burns, electrocution, fire, or injury to persons


 **PLEASE OBSERVE THE FOLLOWING PRECAUTION BEFORE USING THIS PRODUCT**


 Do not open the product to repair or remodel. Only qualified service personnel should service this product. Do not use attachments not recommended by the manufacturer.





 Connect the this product to cold water supply pipe carrying potable water. Do not connect to an industrial water supply line that does not carry water of drinking quality. This will negatively effect hygiene and performance.

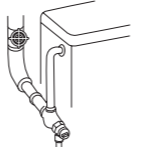



 **About Low Temperature Burns-** A burn occurs when the skin has been exposed to high temperature. However, a low temperature burn may occur when the skin has been in contact with an object at relatively low temperature (about 104 °F). Close supervision is necessary when this product is used by, children or people with sensitive skin.


 If using any chemical toilet bowl cleaner, be sure to wipe up any spills that splash onto Electric Bidet. Do not use benzene or any acid chemical detergent. Only neutral detergent permitted.

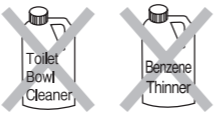


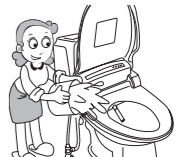
 Do not plug in the power cord before connecting the water supply hose.

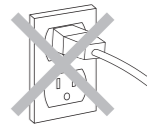



 This product must be grounded. An approved Ground Fault Circuit Interrupt, must be used.





 Do not splash water on this product. Please provide adequate ventilation to avoid damage to electronic components due to high level of humidity. it is necessary to provide adequate ventilation with a fan, open door or open window when bathing.




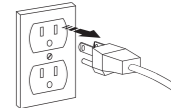
 Do not touch the power cord with wet hands. Do not reach for a product that has fallen into water. Unplug immediately!





 This product must be connected to a standard voltage power source. Do not connect power cord if the cord is damaged or broken. Do not plug the unit into a loose electric outlet.



 Please disconnect the power cord when you clean this product or detach it from toilet bowl. Do not supply electrical power to the unit until the plumbing work has been completed.



 Keep the electrical cord away from heated surfaces to prevent a fire hazard. Also, keep any flammable materials away at all times from the this product.



I-ZEN Bidet

CAUTION



Prohibited

Never operate this product if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the product to a service center for examination and repair



Prohibited

Do not obstruct or put anything into the dry air outlet. Do not apply excessive force to the shower nozzle.



Prohibited

Do not stand or sit on the seat cover lid. It is not designed to support a user's body weight.




Prohibited

Allow the seat and lid to close automatically do not force the seat and lid.



Prohibited

Do not strain or push the connected hose and tube connector; it may cause water leakage. Never drop or insert any object into any opening or hose. Use proper plumbing supplies if any modification is to be made.

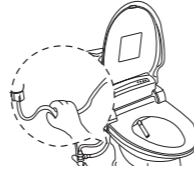

Observe at all times

Use this product only as described in the operating instructions. Never use while sleeping or drowsy. Do not use while bathing.




Observe at all times

Be careful so that this product does not become stained with excrement and urine.



Observe at all times

Drain the water from unit and unplug the power if it is not going to be used for an extended period of time. Refer to page 19

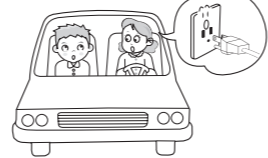

- SAVE THESE INSTRUCTIONS -

2 CAUTIONS BEFORE INSTALLATION

WIRING REQUIREMENT

THIS PRODUCT MUST BE PLUGGED into a regular voltage Grounded GFCI outlet (North America 120V ac, 60Hz / Australia, Europe 220-240Vac, 50-60Hz). The rated power consumption is 660W / 600-700W. In locations where only a standard two-prong outlet is available, it is the responsibility of the user to install a grounded three-prong outlet. Using a higher voltage is dangerous and may result in a fire or other accident. This product is not responsible for damage resulting from the use of it on other than specified voltage, or operating it without proper grounding.



WARNING


This Product must be on a separated circuit. No other appliance should share the circuit with this product. Sharing a circuit could cause the branch circuit fuse to blow or the circuit breaker to trip. Continuous usage under these conditions could result in fire or property damage.

GROUND INSTRUCTIONS

(1) This product has a three-prong grounding plug, which must be plugged into an outlet that is properly installed and grounded. In the event of an electrical short circuit, grounding reduces the risk of electric shock by providing an escape wire for the electric current. If repair or replacement of the cord or plug is necessary, do not connect the grounding wire to either flat blade terminal. The wire with insulation having an outer surface that is green with or without yellow stripes is the grounding wire. Check with a qualified electrician or serviceman if the grounding instructions are not completely understood, or if in doubt as to whether the product is properly grounded.

This appliance have to be earthed.

OPTIONAL GROUND INSTRUCTIONS

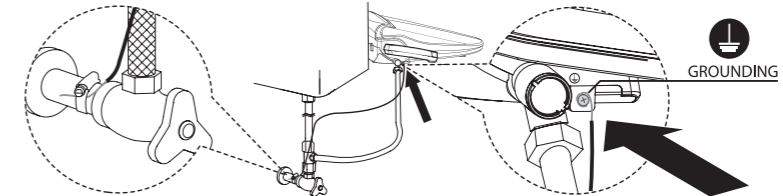

DANGER


IMPROPER GROUNDING CAN RESULT IN A RISK OF ELECTRIC SHOCK. Electric wiring has to be done in accordance with all local codes, or in the absence of local codes, with the National Electrical Codes.

The use of an extension cord is not recommended or advised. If it is necessary to use an extension cord, use only a three-prong extension cord (one to one), and a three slot built-in GFCI receptacle that will accept the plug on this product. The marked rating of the extension cord should be equal to or greater than the electrical rating of the unit (10amp). Consult a qualified electrician or service person if you are not sure whether the unit is properly grounded.


3 IDENTIFICATION

1 MAIN UNIT


IB-8300, 8500

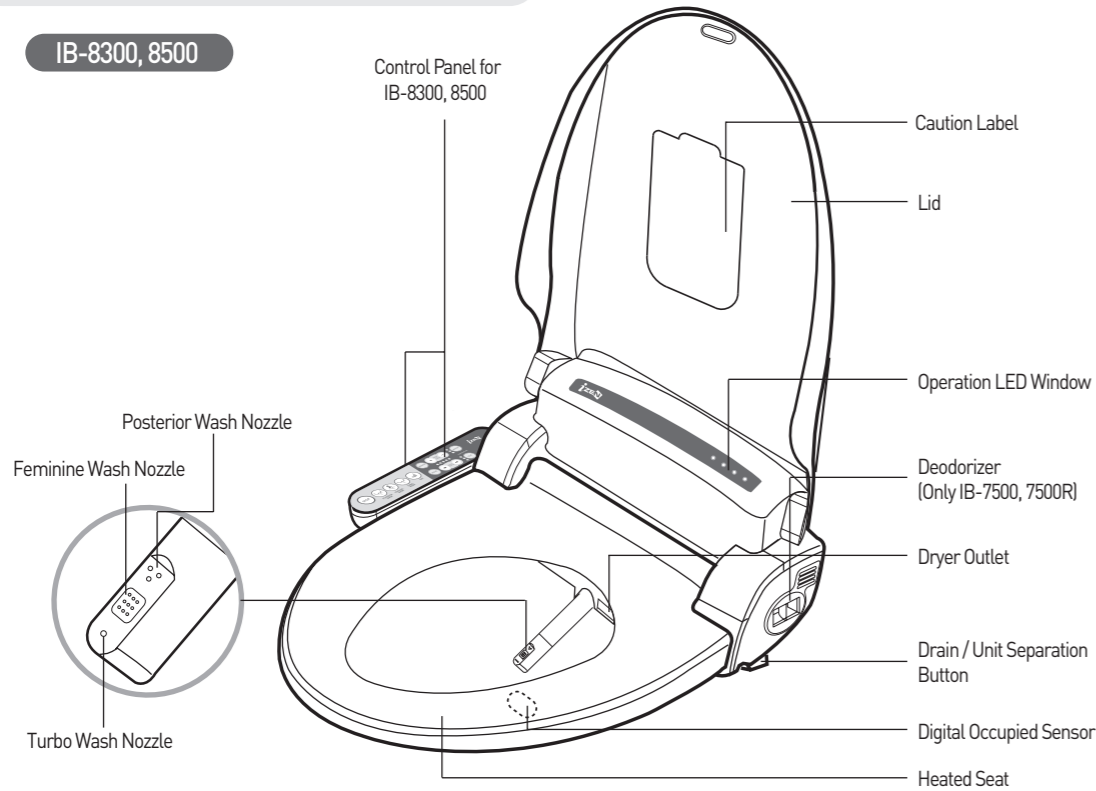

IB-8500R

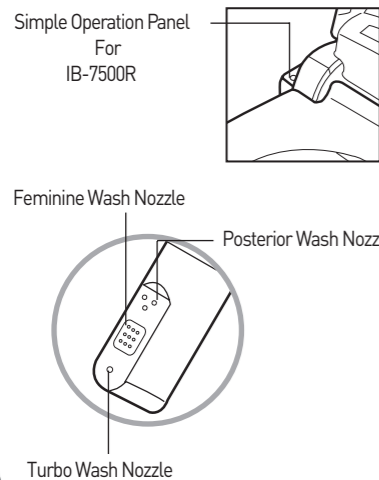


2 ACCESSORIES

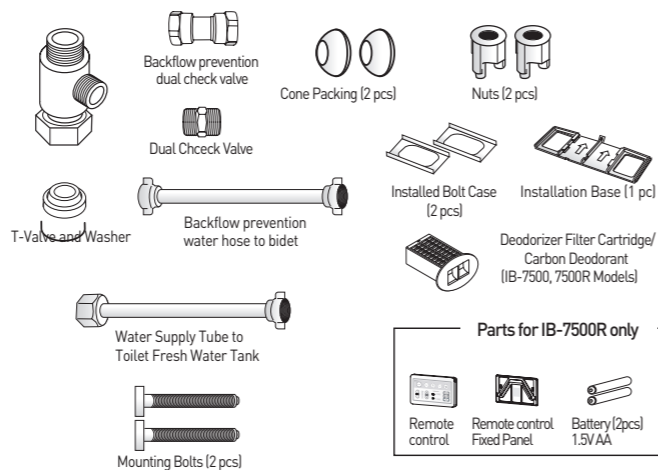


4 OPERATION (BASIC)


IB-8300, 8500 Control Panel

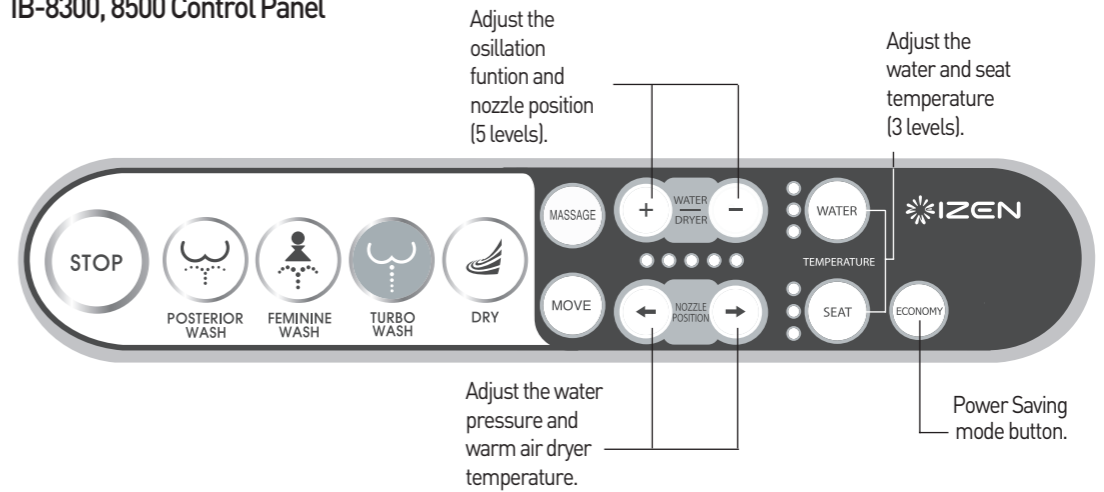

IB-8500R Remote Control

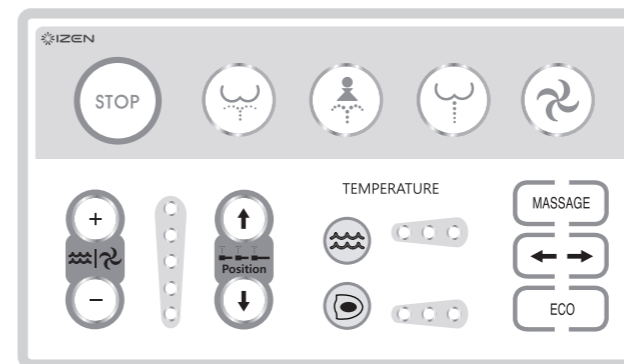

Simple Operation Panel

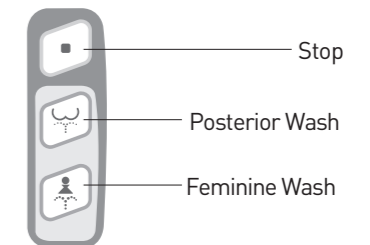

- When you sit on this product, please sit down on the central position in order to activate the sensor.
- After an extended wash, allow water for about three minutes to be reheated.

5 OPERATION (DETAILS)

1 PREPARATION

- Check whether any water leaks.
- Plug the unit into the proper power source and the LED power light will start flashing.
- Wait until the LED power light stops flashing and becomes solid red.
- The water tank will be filled after pushing a "Wash" (♂ or ♀) button.
- When the water tank is filling the buzzer "Beeps" continuously, after the water tank is full the buzzer will end with "One Solid Beep".

2 OPERATION

SIT DOWN ON THE SEAT

- When you sit down on the seat, the ORANGE LED lamp on the control panel will light up.
- When the digital sensor on the seat is activated, the operation LED window on the main unit will light "Occupied sensor" and simultaneously, Deodorizer fan will start to work (IB-8500, 8500R)
- See page 11 for details about changing the seat or water temperature.

STOP

- Press this button to stop all operation.

CLEANSING

CLEANSING 1. (Posterior Wash)

- 1- Press "Posterior Wash" (♂). Automatically, the nozzle comes out and sprays warm water to posterior.
- 2- Additional functions during your Posterior Wash experience are available. See Oscillation, pg.12 and Massage, pg. 13 for details.

CLEANSING 2. (Feminine Wash)

- 1- Press "Feminine Wash" (♀). Automatically, the nozzle comes out and sprays warm water to feminine area.
- 2- Additional functions during your Feminine Wash experience are available. See Oscillation, pg.12 and Massage, pg. 13 for details.

CLEANSING 3. (Turbo Wash)

- 1- Press "Turbo Wash", the nozzle spray is more focused on producing the effect of strong wash, to help stimulate bowel movement.
- 2- Press "Water pressure - High & Low button" to adjust the water pressure.



Observe at all times

CAUTION

DO NOT ALLOW CHILDREN TO USE THE **TURBO WASH** FUNCTION! SUPERVISION IS STRONGLY RECOMMENDED FOR THE ELDERLY OR OTHER PERSONS THAT MAY HAVE LIMITATIONS IN BODY MOVEMENT, ILLNESS OR DISABILITIES!

CLEANSING cont.

CLEANSING 3. (Turbo Wash)

- 1- Press "Turbo Wash" continuously for 3 to 5 seconds. Automatically, the water nozzle comes out and sprays a warm stream of water to the posterior center of the rectum.
- 2- If necessary, adjust "Position" of water stream with the (↑/↓) buttons.
- 3- Press "Water pressure High & Low button" (+/-) to adjust the water pressure to your desired comfort.
- 4- Press the "STOP" (⏹) button when you feel your abdomen is full or wait until the "Turbo Wash" function stops, approximately 1 minutes.
- 5- You will shortly feel the urge to push the water out of your colon, push the water and feces out as completely as you can.
- 6- Relax for a few minutes and be sure you feel your colon is empty and then use the "Posterior Wash" and "Air Dry" function to finish.

WARM AIR DRYER

- 1- Press "Dry" (🌬️), warm, dry air is gently directed to washed area for about 3 minutes.
 - 2- You may adjust the warm air temperature by pressing High (+) and Low (-) buttons.
- ※ This product memorizes the air temperature of prior use.


WATER PRESSURE ADJUSTMENT

- After sitting down on the this product, the LED lamp (●●●●●) for water pressure on the Power Control will light at third or middle level (●●●●●)
- Press high (+), or low (-) to adjust the pressure of the water spray to your desired comfort level


WARM WATER TEMPERATURE AND SEAT TEMPERATURE ADJUSTMENT

- The temperature of water and seat are adjustable to the desired level by pressing the "WATER" (WATER) and "SEAT" (SEAT) button.

WATER TEMPERATURE

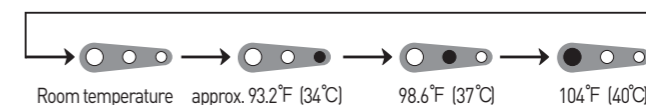

SEAT TEMPERATURE

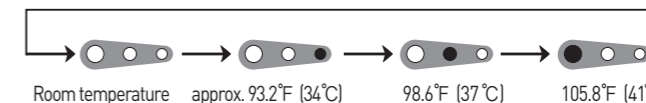

5 OPERATION (DETAILS)


● NOZZLE POSITION ADJUSTMENT

- Nozzle position is adjustable to the desired position by pressing “Front” (↑) or “Rear” (↓) button from 1 to 5 positions. After using this product, the nozzle will return to position 3.

● DEODORIZER

(IB-8500, 8500R Models)

- The deodorizer fan is activated when this product is occupied. When finished, the fan will stop after 1 minute.
- Air current for the deodorizer has 4 adjustable levels. While pressing the “STOP” (⏻) button, press and “WATER” (💧) button to the desired air current level. See diagram:

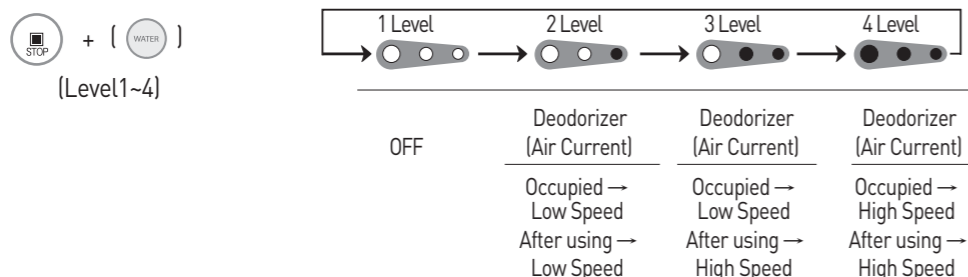

● OSCILLATION (Nozzle Movement Function)

- When using Posterior or Feminine Washing functions, pressing the “MOVE” (↔) button will provide a short-motion, front to back oscillating spray for a much wider cleansing experience. Note: This function will not work during the “Turbo Wash” or “Enema Wash” function.
- You may change the position of the Oscillating spray by pressing the “MOVE” (↔) button again and “Front” (↑) or “Rear” (↓) buttons simultaneously.
- Push the “MOVE” (↔) button again and the oscillation motion will stop automatically.

Auto-Setting of Oscillation Feature

- You can set this function to automatically oscillate whenever the seat is occupied and the Posterior or Feminine washes are in operation.

Step 1: Do not sit on the this product.

Step 2: Press the “MOVE” (↔) button for 3 seconds or until you hear a single Chime.

Step 3: Sit on the this product and proceed with Posterior or Feminine Wash.

Note: In the automatic setting, you may still adjust the position of the Oscillating spray by pressing the “Front” (↑) or “Rear” (↓) buttons.

Canceling the Automatic Oscillation function

Step 1: Do not sit on the this product.

Step 2: Press the “MOVE” (↔) button for 3 seconds or until you hear a double Chime.

● MASSAGE (Rhythm Wash)

- When using the Posterior Wash or Feminine Wash functions, pressing the “MASSAGE” (🌀) button will provide a pulsation spray.
- You may change the pressure of the spray by pressing the “High” (+) button or “Low” (-) buttons.
- Push the “MASSAGE” (🌀) button again and the pulsating action will stop automatically.

Auto-Setting of Massage Feature

- You can set this function to automatically pulsate whenever the seat is occupied and the Posterior or Feminine washes are in operation.

Step 1: Do not sit on the this product.

Step 2: Press the “MASSAGE” (🌀) button for 3 seconds or until you hear a single Chime.

Step 3: Sit on the this product and proceed with Posterior or Feminine Wash.

Note: In the automatic setting, you may still adjust the pressure of the pulsating spray by pressing the “high” (+) or “low” (-) functions.

Canceling the Automatic Oscillation function

Step 1: Do not sit on the this product.

Step 2: Press the “MASSAGE” (🌀) button for 3 seconds or until you hear a double Chime.

● ECONOMY (Energy Save Mode)

- The ENERGY SAVER is not on when the LED light is off.
- Press button once to choose “ECONOMY 1”
- Press button again to choose “ECONOMY 2”
- Press button third time to turn “ECONOMY” off

**The ECONOMY 1: the water temperature will be kept at 90°F (32°C)*

**The ECONOMY 2 (SMART MODE): It will lower the water temperature to 90°F (32°C) when it is not being used, and raise the water temperature back to normal level during the time of use.*

It takes about several days / a few days for the unit to “learn”, and “remember”, the usage pattern of the user(s)

● NOZZLE CLEANING (Automatically)

- This product has an automatic nozzle self-cleaning function that cleans the nozzle before and after each use.

● NOZZLE CLEANING (Manual)

- Nozzle can be cleaned manually by pressing “Posterior Wash” (🚿) button for 3 seconds without being occupied on the seat and nozzle will come out without spraying water. After cleaning the nozzle, Press “Stop” (⏻) button to return nozzle to stand-by mode. (🚿, 🚿: in Simple Operation panel -Remote Model)

● CHIME SOUND ON/OFF SET

- While sitting on the this product, press “MOVE” (↔) button for 3 seconds and you can hear the chime sound change to ON or OFF. (Remote model doesn't have off function, IB-7500R)

6 INSTALLATION


⚠ CAUTION BEFORE INSTALLATION

- Do not plug in before installation.
- If you need an electric power system or extension, please consult a qualified electrician.
- The installation must comply with the Australian plumbing installation code AS/NZS 3500
- A product that is listed on the WaterMark Product Database and is marked in accordance with the WaterMark Certification Scheme is recognised by authorities having jurisdiction as being authorised for use in a plumbing or drainage installation. This is because the product complies with the applicable product specification. The installation of an authorised product must meet the requirements of the PCA. Where the PCA does not contain installation requirements applicable to the authorised product, acceptance of the installation is at the discretion of the authority having jurisdiction.

STEPS

1 Turn Off Water.

- Shut off the water supply to the existing toilet tank by closing the water supply valve. Drain the toilet tank completely. Remove the water supply hose to toilet tank.

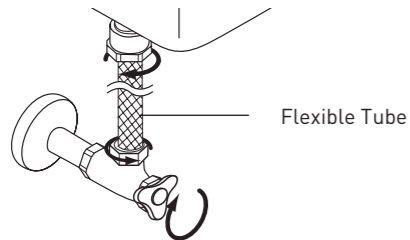


2 Removing the existing seat and lid.

- By using a wrench, (Spanner) or other proper tool, release the nut on both sides of existing toilet to remove the toilet seat and lid.




3 Connect T-Valve.

- Remove existing supply tube from fresh tank
- Water Supply Valve is standard 1/2" in size. (If it is 3/8", use the 1/2" to 3/8" conversion nut kits with packing before connecting the Water Supply Valve.)
- Install T-Valve on top of Water Supply Valve.

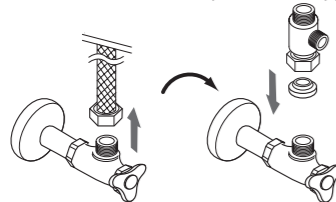

4 Water Supply to Tank Assembly.


- One flexible tube connects from the top of the T-Valve to the toilet tank.

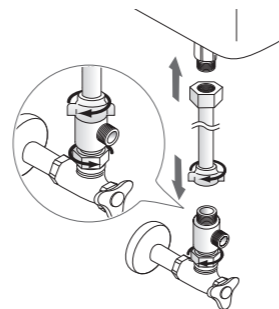

※ Check whether any water leaks.


5 Installation Base Assembly.

1. Put the 2 Bolts into the 2 Installed Bolt Cases
2. Place this Bolt assembly into the Installation Base as shown.
3. Guide Arrows on the Installation Base should face up and towards the Toilet Tank.
4. Place the Installation Base and guide the Bolts into the 2 holes on the toilet bowl.

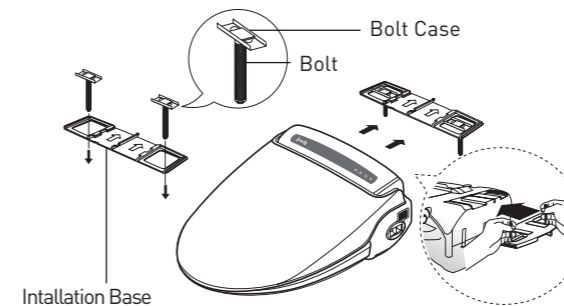


6 Bolt and Nut Assembly.

1. Fit the cone packing and plastic washers onto the bolts from below the toilet bowl as shown.
2. Loosely tighten nuts by hand.

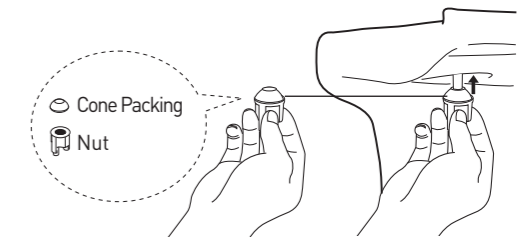

7 Assembly.


1. Slide this product onto the installation Base centering the unit with the guide arrows on the Base.
2. You will hear a click once this product is securely mounted to the Installation Base.
3. The product will feel a little loose on the toilet bowl. Adjust the fit from front to back and side to side for balance.
4. Tighten the nuts on both bolts securely.

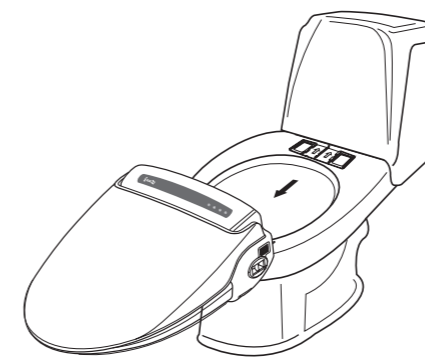

※ Check whether any water leaks.

8 Water Supply Tube Assembly.


1. Connect the flexible tube from side of T-Valve to this product.
2. Open the water valve.
3. Check for leaks.
4. If there are no leaks and the surrounding area is clean and dry, connect the power cord to a ground protected power outlet.

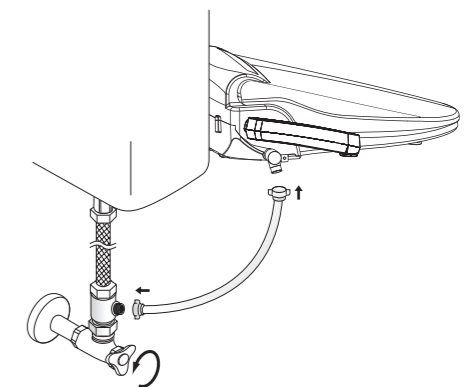

※ Customers should use the new hose set provided with the product. Do not use used set again.


6 INSTALLATION

- 9 Deodorizer Installation.**
Insert the deodorant cartridge into the unit.
(IB-8500, 8500R)

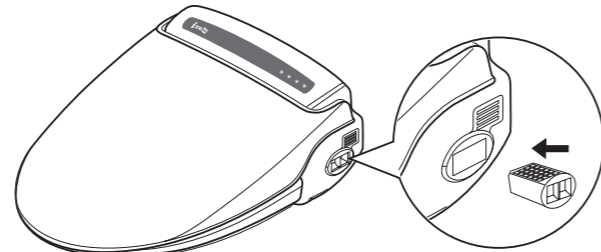


How to change the deodorant catalyst

- 1**
- Check the deodorant package.
 - Follow the below photos to change the deodorant.

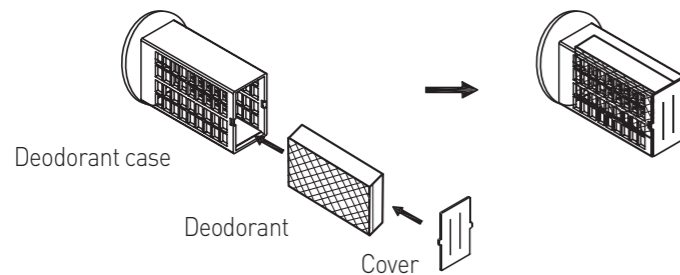


- 2** Insert the new deodorant case into the unit.

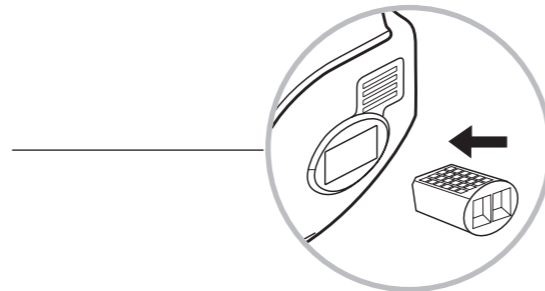


- 10 Installation of Remote Control on wall or convenient surface. (IB-8500R)**

1- Mount Fixed Panel to wall location.
Take off the cover on the adhesive on Remote Control Fixed Panel or use adhesive tape in order to adhere Remote Control Fixed Panel on wall.

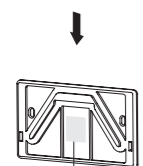

2- Then slide Remote Control on to Remote Control Fixed Panel.

Adhesive Tape

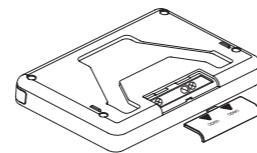



Place 1.5V(AA) batteries properly and shut the cover

7 SEPARATION OF UNIT FROM THE TOILET BOWL

- 1** Disconnect the power plug
- 2** Shut off the Water Supply Valve.
- 3** Push the button on the right side of the unit to completely drain the water from water tank.
- 4** Separate the unit from toilet as following photos.

•When you pull the unit while pushing the drain button, the unit will separate from the installation base.

! Do not turn unit upside down.

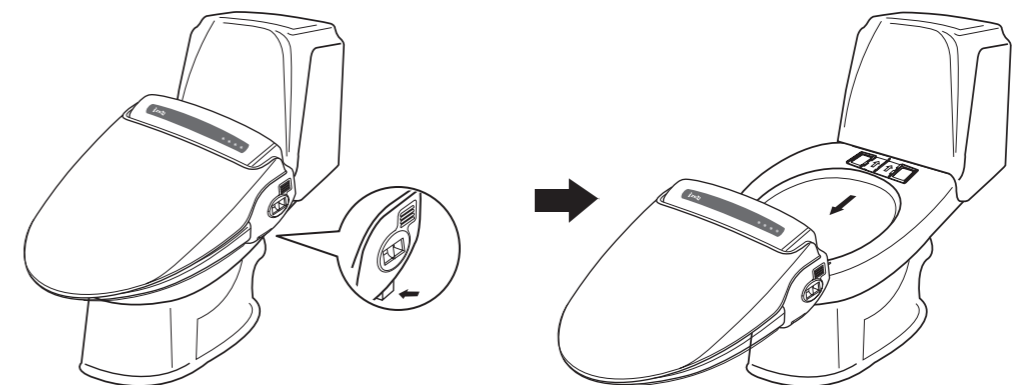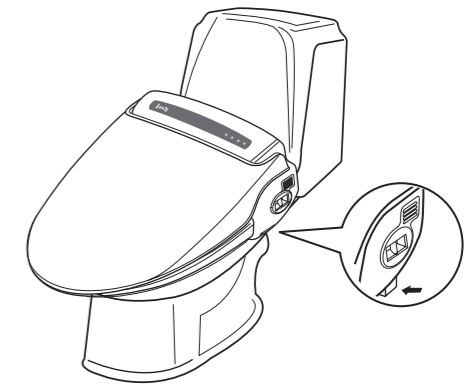

8 MAINTENANCE

CLEANING THE NOZZLE

- When the "Posterior Wash (👤/👤)" button is pressed for 3 seconds without occupying the seat, the nozzle can be cleaned with a soft toothbrush.
(👤: in Simple Operation panel -Remote Model)
- After cleaning the nozzle, push the "Stop (🛑, 🛑)" button once again, and the nozzle retracts itself.
(🛑: in Simple Operation panel -Remote Model)





CLEANING THE BODY OF UNIT

- When you clean this product, do not use "thinner", "benzene" or toilet bowl cleaner. It may cause damage to the surface of unit.
- Use a wet cloth when you clean this product.
- Do not spray water directly on to this product.
- When you clean this product, disconnect the power plug.

CLEANING THE STRAINER

This product will not operate properly if the strainer is clogged. Please clean the strainer regularly.

- 1) Close the water shut off valve.
- 2) Flush the toilet to empty the water reservoir.
- 3) Unscrew the water filter nut.
- 4) Clean the strainer with a small brush or water.
- 5) Re-screw the water filter nut.
- 6) Open the water shut off valve.

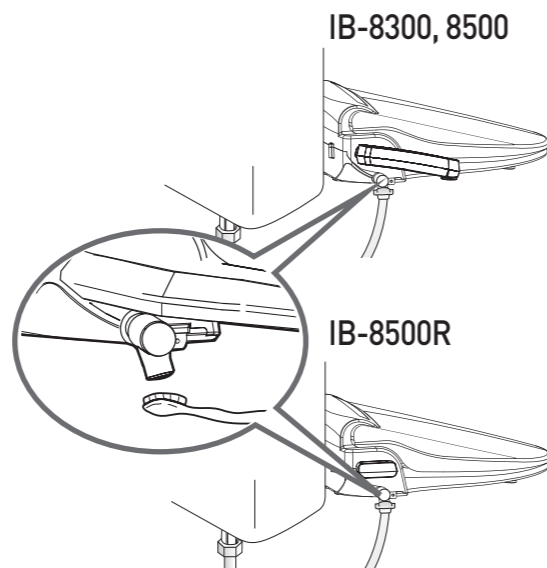
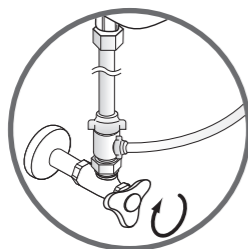


9 WATER DRAINING

DRAINING THE WARM WATER TANK IN THE UNIT

If you do not intend to use this product for 14 days or longer, drain the water from the warm water tank of this product.

- 1) Disconnect the power plug.
- 2) Close the water shut off valve.
- 3) Press the drain lever until the water is completely drained from the unit's water tank.
- 4) To re-supply water after draining, open the water shut off valve.

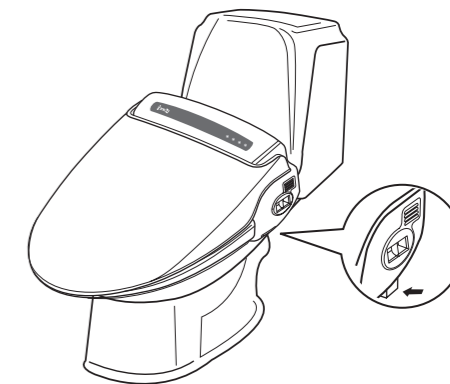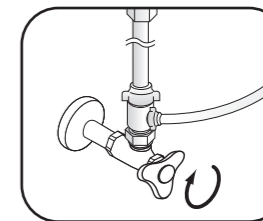

Do not drink the drained water.
Do not insert finger into the drain hole.
Before draining, unplug the power cord and wait 30 minutes until the unit has cooled down.

MAINTENANCE FOR WINTER SEASON

- 1 Check the power cord whether it is plugged in or not.
- 2 Keep the water temperature on "Middle or High Level" to prevent water from freezing.
- 3 If you do not intend to use the unit for 14 days or longer, disconnect the power plug and drain the water.

10 TROUBLE SHOOTING

Trouble	Cause	Remedy
The washing water does not spray	<ol style="list-style-type: none"> The water tank is not filled. The shut off valve is closed. Freezing inside the main unit or the water supply pipe. 	<ol style="list-style-type: none"> Wait for about one minute. Press the desired washing button once again. Turn on the shut off valve to open. Heat the room (for about two hours). Operating the unit while parts are frozen will cause a malfunction.
The unit is not operating	<ol style="list-style-type: none"> Disconnected the power cord. The seat is not occupied. The battery of remote controller is exhausted. (Only IB-7500R model) 	<ol style="list-style-type: none"> Connect the power cord. Check the power supply system in your house. Occupy the seat (The unit has an occupied sensor). Replace the battery.
Water is leaking	<ol style="list-style-type: none"> Check the water supply system 	<ol style="list-style-type: none"> If water is leaking from the main unit, call the service center or the place you purchased.
The washing water pressure is weak	<ol style="list-style-type: none"> The water pressure level is set to "LOW". The strainer is clogged. 	<ol style="list-style-type: none"> Set the water pressure to "HIGH". Clean the strainer.
The washing water is not warm	<ol style="list-style-type: none"> The water temperature is set to "LOW". Continuous operation has depleted the supply of warm water. 	<ol style="list-style-type: none"> Set the water temperature to "HIGH". Wait for about three minutes.
The seat and dryer is not warm	<ol style="list-style-type: none"> The seat & dryer temperature is set to "LOW". 	<ol style="list-style-type: none"> Set the seat & dryer temperature to "HIGH".

11 SPECIFICATIONS

Specification	IB-8300	IB-8500	IB-8500R	
Power Rating	Australia or Europe: 220 - 240V~, 50/60Hz, 600 - 700W America or Canada: 120Vac, 60Hz, 660W			
Water inlet type	Type	Direct Inlet flow type		
	Water inlet pressure	0.04 ~ 0.74 MPa (5.69 psi ~ 106.7psi)		
Power cord	(1.2m [4 feet]: AMERICA) / (1.8m:Australia or Europe)			
Wash Device	Posterior Wash	Max. 1.20/min (Max. 0.32 GPM)		
	Feminine Wash	Max. 1.20/min (Max. 0.32 GPM)		
	Water Pressure Adjustable	5 Levels		
	Warm Water Temperature	Room Temp - 40 °C (Room Temp. 104°F)		
	Heater	600 W		
Safety Devices	Thermal fuse, Thermal switch, Float switch			
Heated Seat	Temperature	Room Temp ~ 41 °C (Room Temp. ~ 105.8 °F)		
	Heater	55 W		
	Safety Device	Thermal fuse		
Warm Air Dryer Device	Adjustable Dry Temperature (Room Temp~45°C)	○	○	○
	Heater (250W)	○	○	○
	Safety Device (Thermal fuse, bimetal)	○	○	○
Turbo Wash Function	○	○	○	
Deodorizer	-	○	○	
Remote control	-	-	○	
Inlet water temperature	3°C~ 35°C (37.4°F ~ 95°F)			
Ambient Temperature	3°C~ 40°C (37.4°F ~ 104°F)			
Dimension	Elongated(Control)	Round (Control)	Elongated (Remote)	Round (Remote)
	W : 495mm (19.4 inch) L : 525mm (20.6 inch) H : 185mm (7.28 inch)	W : 495mm (19.4 inch) L : 493mm (19.4 inch) H : 185mm (7.28 inch)	W : 475mm (18.7 inch) L : 525mm (20.6 inch) H : 185mm (7.28 inch)	W : 475mm (18.7 inch) L : 493mm (19.4 inch) H : 185mm (7.28 inch)
Weight (kg)	5.3kg	5.4kg	5.2kg	

12 WARRANTY

IZEN CO., LTD. One Year Limited Warranty

- The IZEN CO., LTD. warrants the Shower Toilet Seat to be free from manufacturing defects under normal and intended use. This warranty is extended only to the original retail buyer and residential location for a period of (1) years from the date of purchase.
- The IZEN CO., LTD. obligations are limited only to the repair or replacement of the Shower Toilet Seat that we find to be defective provided that the Shower Toilet Seat was properly installed and used according to the instruction manual included.
- The IZEN CO., LTD. will not charge for Labor or parts in connection with the repair or replacement of the Shower Toilet Seat.
- The IZEN CO., LTD. will not be responsible for any consumer costs related to the removal, return to The IZEN CO., LTD. and /or reinstallation of the Shower Toilet Seat.
- The IZEN CO., LTD. will not be responsible for any consumer loss related to incidental, consequential damages or expenses incurred by the purchaser.
- Not covered by this warranty are:
 - Loss or damage from fire, flood, freeze, electrical storm or any other natural calamity or act of God.
 - Defects or malfunctions resulting from failure to properly install, operate and/or maintain the Shower Toilet Seat to residential water toilets in accordance with the enclosed installation manual.
 - Loss or damage from misuse, abuse or negligence.
 - Damage resulting from unauthorized alterations, attachments and/or repairs.
 - Loss or damage from installing the unit in a harsh or hazardous environment and/or an environment including animals, mice or insects.
 - Damage caused from excess moisture or from sediments or foreign matter contained in the water system.
 - Damage from not being installed in accordance with all local codes, ordinances and good trade practices.
 - Color fading due to an installation in the wrong environment.
 - Loss or damage from consumable chemicals such as chlorinates (found in public water supplies) or from cleaning agents warned about in the installation manual.
 - Damage to the toilet bowl that the Shower Toilet Seat is installed to.
- The IZEN CO., LTD. will only warrant products with returned registration cards or from on-line registration with proof of purchase within 30 days of the original purchase date.
- This warranty is also void if the "CAUTION" label is removed from the inside of the Shower Toilet Seat cover.
- This limited warranty gives you specific legal rights and you may also have other rights that may vary from state to state.

This is the IZEN's exclusive limited warranty.

13 INFORMATION TO THE USER (For America)

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, it may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

WARNING

Changes or modifications not expressly approved by the manufacturer could void the user's authority to operate the equipment.

Note : The manufacturer is not responsible for any Radio or TV interference caused by unauthorized modifications to this equipment. Such modifications could void the user's authority to operate the equipment.

WARRANTY REGISTRATION

To ensure your warranty protection, you must complete and return this card along with a copy of your receipt to The IZEN CO. LTD. within 30 days after Date of Purchase.

Model _____ Serial Number # _____

Purchaser _____ Date of Purchase _____

company _____

Address _____

City _____ Country _____

Phone _____ E-mail _____

Dealer(Company) _____


Dealer Code _____

Address _____

MAIL TO :

Australia IZEN

Office : 4 University Place, Clayton VIC 3168
 CS: 0433 104 670
 Fax: 03 9543 6704
 Email: izen.australia@gmail.com



www.izenseat.com
www.izenbidet.com.au

Australia IZEN

Office : 4 University Place, Clayton VIC 3168
CS: 0433 104 670 (Australian Landline Only)
M.03 9548 8117 (Mobile and outside Australia)
Fax: 03 9543 6704
E-mail: izen.australia@gmail.com